

Quick Start Placemat

Please read **ALL** instructions carefully before using this placemat.
Scan QR Code to watch a video on how to run a test.

91279 Rev. A 07/21

ReliOn™
A1C Self Test System



Match Lot Numbers



Make sure lot numbers on analyzer and pouches match before opening.



If lot numbers do NOT match call Customer Service at 1-855-776-0662.

5 Mix



Shake vigorously 6-8 times (about 5 seconds). This will mix the blood with the solution.



Stand shaker on table while preparing cartridge.

6 Open Foil Cartridge Pouch



After blood collection, tear foil pouch to open at the notches on the sides.



Insert cartridge into analyzer immediately and use within 2 minutes.

1 Open Plastic Shaker Pouch




Do NOT open foil pouch until Step 6.

2 Stick Finger



Wash hands or clean with alcohol and dry before testing.

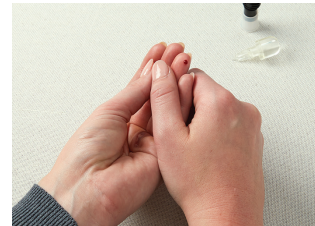

Gently squeeze finger to get adequate blood.

7 Insert Cartridge



"Click" test cartridge into place.



Analyzer and test cartridge codes must match.

8 Prepare Shaker



Remove shaker base.



WAIT for SMPL to display



Ready for shaker

3 Collect Blood



Gently touch blood collector to top of blood drop to fill.

Make sure blood collector is adequately filled.

4 Insert Blood Collector



Fully insert blood collector into shaker body.




Push hard while twisting to fully insert.

10 5 Minutes to Results

- Analyzer will cycle "QCOK," your result, and # of tests left for 15 minutes and will turn off automatically.
- If "QCOK" is not displayed, refer to the troubleshooting section in the Overview & Quick Reference Guide.
- Record your result.
- If the analyzer displays "01TL," save for next test.
- If analyzer displays "00TL," you used the last test.
- Dispose of cartridge in your household waste.




Customer Service 1-855-776-0662.

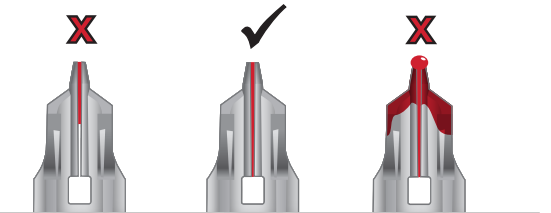

Too Little
Add more blood

Just right

Too Much
Wipe away excess

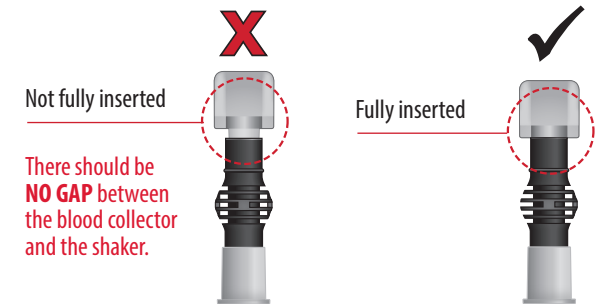

Not fully inserted

There should be **NO GAP** between the blood collector and the shaker.

Fully inserted

9 Dispense Sample into Cartridge



- Ensure analyzer is on level surface.
- Push down completely to dispense diluted sample and remove in a continuous motion.
- Remove shaker from cartridge quickly.
- "RUN" will appear.



Do not handle analyzer again until test is complete!



Testing in progress



Display counts down